

Operation Manual

Adapter for Juniorplattform SA-2



JUNIORPLATTFORM

This operation manual be effective only with the operation manual of stairclimber type SA-2!

FIXING THE ADAPTER TO STAIRCLIMBER SA-2

1) Place adapter (U-opening to front) on the column with perforated holes.

2) Fix adapter at 3rd hole (from below) by thread pin (2 pieces M 12 x 25) at the perforated holes.

Finger tighten stud bolt.

3) Hang in the Junior Platform from above into the adapter.

The Junior Platform now rests with two rubber-bumpers on bottom backside of the wheelchair adapter.

Adjust both supports between the ramps to make it possible to drive over by wheelchair. Supports have to stand on the ground to support the ramps.



Fix the wheelchair with the clamps (see operation manual, (chapter 3, point 6).

Fix the handicapped person with the safety-belt (see chapter 3, point 7)

Fix the wheelchair with both safety-belts which are assembled on the Junior Platform.

Pay attention that the capacity of the stairclimber SA-2 will be reduced by the weight of the Junior Platform (16 kg).


That means the max loading capacity of the stairclimber together with the Junior Platform is only 114 kg (person plus wheelchair)!

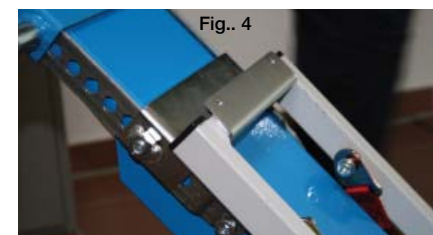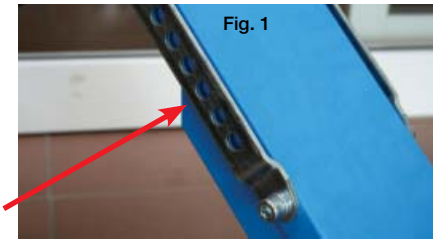

NOTES



M. BARTELS
REHA IM- & EXPORT GMBH

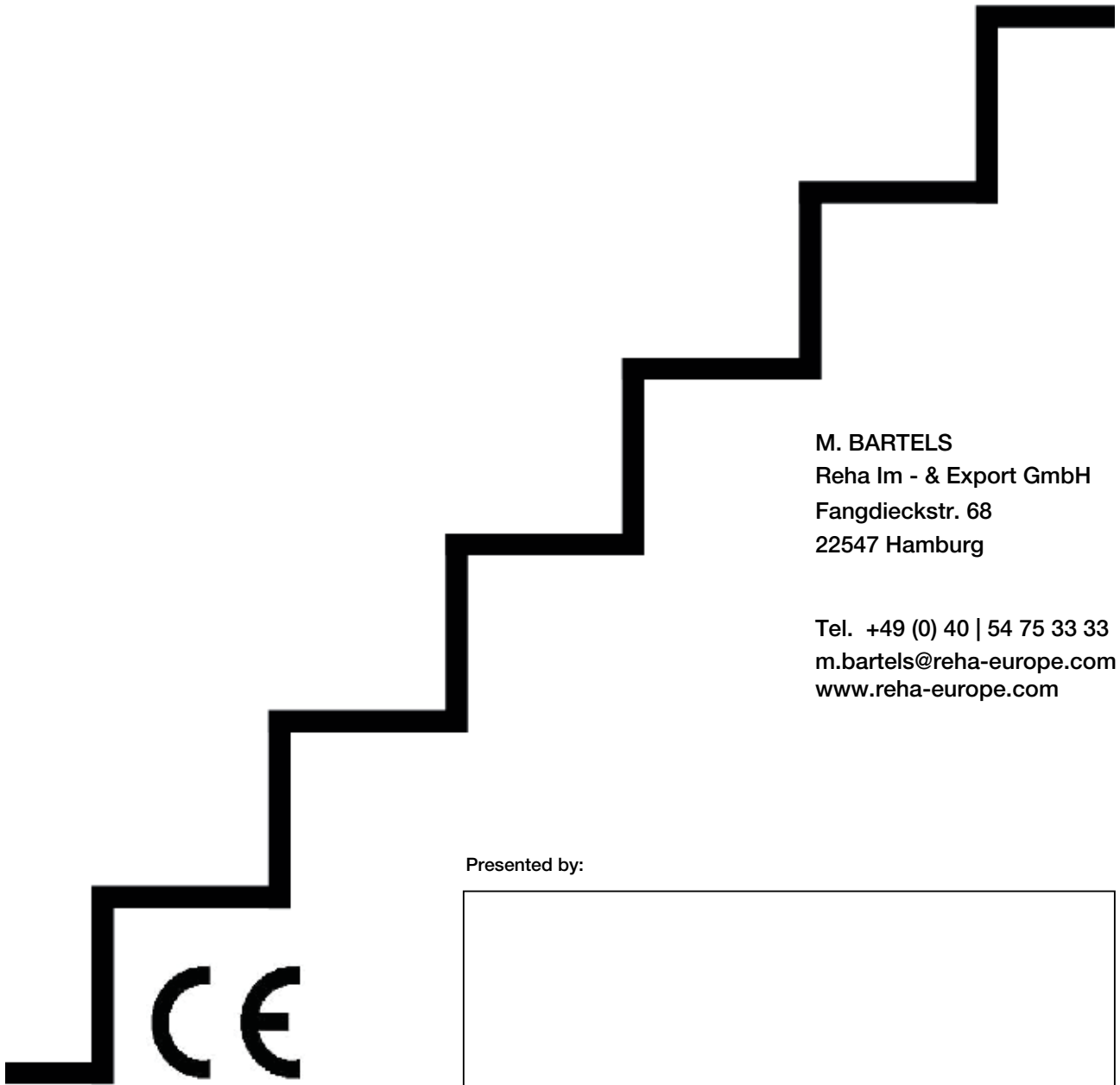

M. BARTELS
Reha Im - & Export GmbH
Fangdieckstr. 68
22547 Hamburg

Tel. +49 (0) 40 | 54 75 33 33
m.bartels@reha-europe.com
www.reha-europe.com

Presented by: